## EN Increasing the retention of the orthodontic expanders cementation

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7-year-old male patient.

### COMPLAINT

The patient's mother complained of "a crooked bite in the front and teeth breaking".

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### EN CLINICAL/RADIOGRAPHIC EXAM

Facial evaluation, clinical evaluation and orthodontic documentation analysis were carried out, which included models in plaster, photos, x-rays and cephalometric analysis. The diagnosis was of anterior crossbite and of teeth 16/46 and 26/36, in mixed dentition.

### TREATMENT

Rapid maxillary expansion with a McNamara expander, aiming at the correction of the posterior crossbite, followed by the reverse traction of the jaw, with a facial mask for correction of the anterior crossbite.

// For teeth with short and/or expulsive clinical crowns, the resin cementation is a good alternative that increases the retention of the expanders. When removing the braces, residues are easily identified as the resin is blue. //

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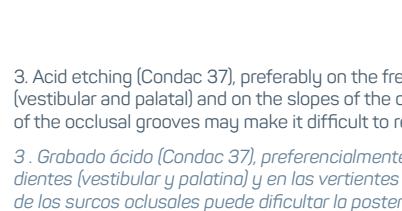
Profa. Dra. Renata Pilli Jóias

STEP BY STEP | PASO A PASO

1. Initial aspect of dental arches showing anterior and posterior crossbite.  
2. Prophylaxis for removal of biofilm and acquired film, which provides better adhesion.

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3. Acid etching (Condac 37), preferably on the free sides of the teeth (vestibular and palatal) and on the slopes of the occlusal surface. Etching of the occlusal grooves may make it difficult to remove the braces.

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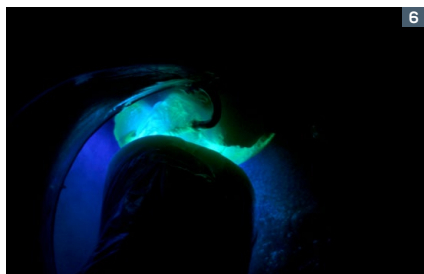
4a and 4b. Application of Ortho Bite inside the acrylic covering of the teeth, in a small quantity, to minimize the work of removing the excess resin.

5a and 5b. With relative isolation, the braces are installed on the teeth and the excess is removed.

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6. Light curing should occur for 40 seconds on each side.

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FGM MATERIALS USED | MATERIALES FGM UTILIZADOS

LIGHT-CURING COMPOSITE FOR OCCLUSAL STOPS AND BAND CEMENTATION  
**Ortho Bite**

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