FINAL

INITIAL | INICIAL

CASE HIGHLIGHTS | DESTAQUES DEL CASO

When taking the shade, it is essential to record canines separately, since these teeth have greater color saturation.

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The time of daily use varies according to the concentration: for 7.5% hydrogen peroxide, the recommended time is an hour and a half.

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After 3 weeks of continuous use of the gel, shade similar to B1 was homogeneously obtained.

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EN

Supervised take-home dental whitening with hydrogen peroxide

AUTHORS: Bruno Lippmann, Raffaella Ronchi Zinelli and Rafael Cury Cecato

Female patient, 23.

COMPLAINT

Yellowish teeth.

INITIAL APPEARANCE

Harmonious smile showing the possibility of whitening, which would improve aesthetics for the patient.

CLINICAL/X-RAY EXAM

Healthy oral condition, teeth showing shade A3.5 (canines) and A3 (other teeth).

TREATMENT

The technique of supervised take-home whitening was carried out, with 7.5% hydrogen peroxide, used for an hour and a half a day for 3 weeks.

“ Hydrogen peroxide for take-home whitening produces excellent results when well indicated. Noticing the results soon during the first days of use of the gel motivates the patient to continue the treatment correctly, and the reduced time of daily use is a benefit that allows the treatment to work even in the busiest schedules. ”

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DR. BRUNO LIPPMANN

FGM MATERIALS USED

TAKE-HOME DENTAL BLEACHING GEL
(HYDROGEN PEROXIDE)
White Class 7.5%

PLATES FOR DENTAL TRAYS
Whiteness Plates for Custom
Dental Trays 1mm

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