EN In-office teeth whitening: practicality, effectiveness and comfort. Is it possible?

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39-year-old female patient.

COMPLAINT

Dissatisfaction with teeth coloring.

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EN CLINICAL/RADIOGRAPHIC EXAM

Absence of cavities and cervical lesions, healthy periodontium. Patient reported caring for the teeth by brushing and cleaning.

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TREATMENT

Dental whitening with 35% hydrogen peroxide (Whiteness HP AutoMixx) in 2 sessions of 45 minutes each with a 7-day interval and sensitivity control by using Desensibilize KF 2%.

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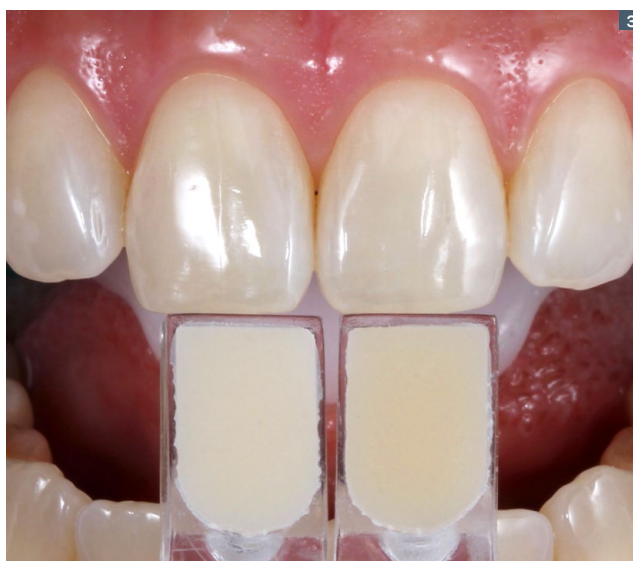
// Whiteness HP AutoMixx is a 35% hydrogen peroxide-based whitening gel with pH stability and that can be applied for 50 minutes with no need for changing. Due to its double-syringe presentation and consequent self-mixing, the application is done easier. It also has color stability during contact with dental tissues and presence of calcium, contributing to the maintenance of dental enamel integrity during the whitening process. //

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7. Final shade evaluation after 2 clinical sessions, compatible with A1 of the VITA shade guide.

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8. Sonrisa final.

FGM MATERIALS USED | MATERIALES FGM UTILIZADOS

IN-OFFICE WHITENING GEL
(HYDROGEN PEROXIDE)
Whiteness HP AutoMixx

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LIGHT-CURING GINGIVAL BARRIER
Top dam

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