FINAL

INITIAL | INICIAL

CASE HIGHLIGHTS | DESTAQUES DEL CASO

Oral health and yellow teeth. The intraoral photograph shows the saturation of the color on the teeth, especially the canines.

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Whitening gel applied along with gingival barrier. During the in-office sessions, the gel remained in contact with the teeth for 40 minutes.

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Demonstration of the use of the whitening gel in the take-home mode with trays.

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Appearance at the end of the treatment, showing the change of shade of the canines that were previously quite saturated.

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EN

Whitening of vital teeth by the mixed technique

AUTHOR: Paulo Henrique Fagundes, Maria Goretti Brito, Mônica Rosas, Pricila Facchinetti, Fillipe Cardoso and Leonardo Muniz

Female patient, 24.

COMPLAINT

Yellowish color of the teeth.

INITIAL APPEARANCE

Teeth with moderately saturated yellow color.

CLINICAL/X-RAY EXAM

Teeth and gums were healthy and there was no need for clinical intervention prior to whitening.

TREATMENT

The proposed treatment was whitening by using the mixed technique, with two sessions performed for application of Whiteness HP Blue 35% gel for 40 minutes and 3 weeks using Whiteness Perfect 16% gel for two hours a day.

“ We, from the Nature Design, have been using FGM’s whitening products for a very long time. In addition to the excellent aesthetic results, tooth sensitivity is moderate to absent in most patients. ”

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EQUIPO NATURE DESIGN

FGM MATERIALS USED

TAKE-HOME DENTAL BLEACHING GEL
(CARBAMIDE PEROXIDE)
Whiteness Perfect 16%

IN-OFFICE DENTAL BLEACHING GEL
(HYDROGEN PEROXIDE)
Whiteness HP 35%

PLATES FOR TRAYS
Whiteness Plates for Custom
Dental Trays 1mm

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